

Human Keratin Matrix

2x2 cm, 4x4 cm, 6x6 cm, 10x10 cm, 12x2 cm




Consult Instructions
for Use



Sterilized using
irradiation



Manufacturer

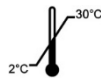

Temperature
limit



Do not
reuse



MR
safe



Use by
date



Lot
number



Kosher
Certified

 ProgenaCare Global, LLC, 2275 Northwest Pkwy SE, Suite 170, Marietta, GA 30067

Federal (USA) law restricts this device to sale by or on the order of a licensed physician.

Instructions

Single use. Do not use contents if pouch is opened or damaged. Store between 2° and 30°C.

Product Description

ProgenaMatrix™ is a clear keratin matrix designed to assist wound repair by providing a moist wound healing environment. It is packaged in a sterile moisture-proof, peel-open pouch containing water and propylene glycol.

Indications for Use

ProgenaMatrix™ is indicated for dry and exuding partial and full thickness wounds such as: pressure (stage I-IV) and venous stasis ulcers, ulcers caused by mixed vascular etiologies, diabetic ulcers, donor sites and grafts, first and second degree burns, superficial injuries, cuts, abrasions and surgical wounds.

ProgenaMatrix™ is not intended to be used on third degree burns.

Directions for Use

1. Prepare wound area using standard methods to ensure wound is free of debris and nonviable tissue.
2. Using aseptic technique, remove matrix from its peel-open package and trim if necessary. Immediately apply matrix onto wound to ensure ProgenaMatrix™ remains optimally hydrated.
3. For proper orientation, apply ProgenaMatrix™ onto wound using embossed lettering as guide. The side reading "UP" should be facing up and away from wound surface.
4. Place matrix in intimate contact with wound surface, extending beyond wound margins.
5. Apply an appropriate secondary dressing to ensure that ProgenaMatrix™ is protected and remains sufficiently hydrated.
6. Reapply matrix as clinically needed.

NOTE: These directions are only guidelines and should not supersede institutional protocols or professional clinical judgement concerning patient care.

Precautions/Warnings

- Prior to use, read all package insert instructions and precautions.
 - Do not use if the product package is damaged or opened.
 - Do not use this device without the direction of an authorized healthcare professional.
- User should advise patient of appropriate precautions relevant to his or her case.
- Control any bleeding at wound site before applying matrix.
- Discontinue use if signs of maceration, odor, or purulence are present, which may be indicative of infection.
- Must be used prior to expiration date.
- Do not re-sterilize.
- Discard any unused material.
- ProgenaMatrix™ is a fully hydrated matrix and does not absorb wound exudate; instead, it manages wound exudate through moisture vapor transmission and, if needed, fenestration.

For more information about ProgenaMatrix™, see www.progenacare.com.